


Quark Antigen Test Kit components:

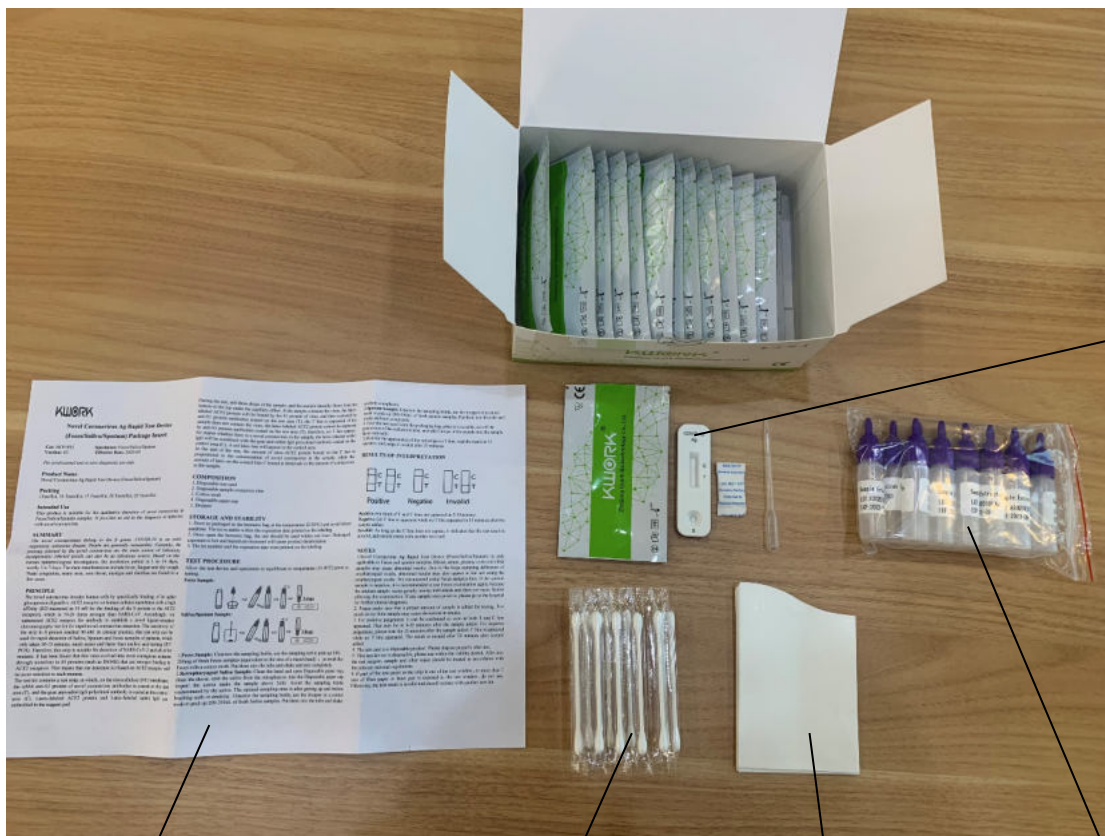

Test Card

Instruction

Swab

Paper Cup

Sample Bottle

Instructions for performing the Quark Antigen Test:

1. Wash hands with soap in the correct medical manner.
2. Wear gloves and mask before each testing.
3. Place contents of bag onto table.



4. (For saliva/sputum) Open Disposable paper cup, clean the throat, spilt the saliva/sputum from the retropharynx into the Disposable paper cup (repeat the action make the sample above 2ml).

(For feces) Open Disposable paper cup, collect feces into the Disposable paper cup (repeat the action make the sample above 2ml).



-
5. Unscrew the sampling bottle, use the dropper or a cotton swab to pick up 200-300uL of fresh samples and then put into the sampling bottle, shake and mix completely.






6. Screw the top back on the sampling bottle.




7. Place test card onto table.

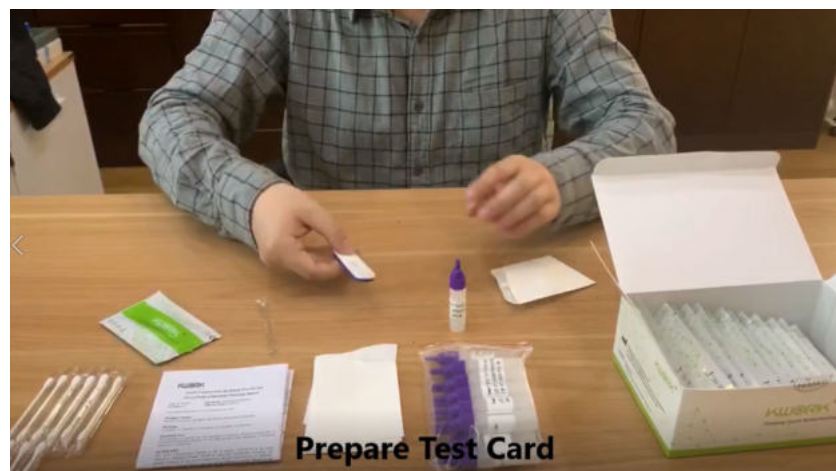

8. Cut the protrusion (top) of the sampling bottle.



9. Add 3 drops of sample vertically into the sampling hole of test card.


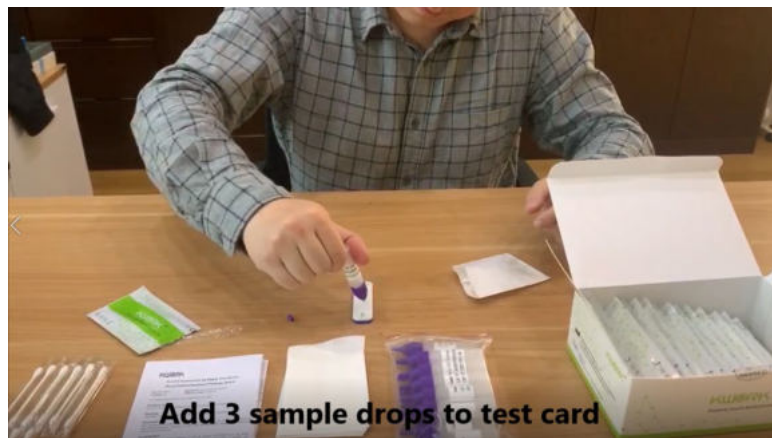


10. Begin timer of 15 minutes.



11. Red line should appear – consult notes for interpretation of result.

12. Read result at 15 minutes

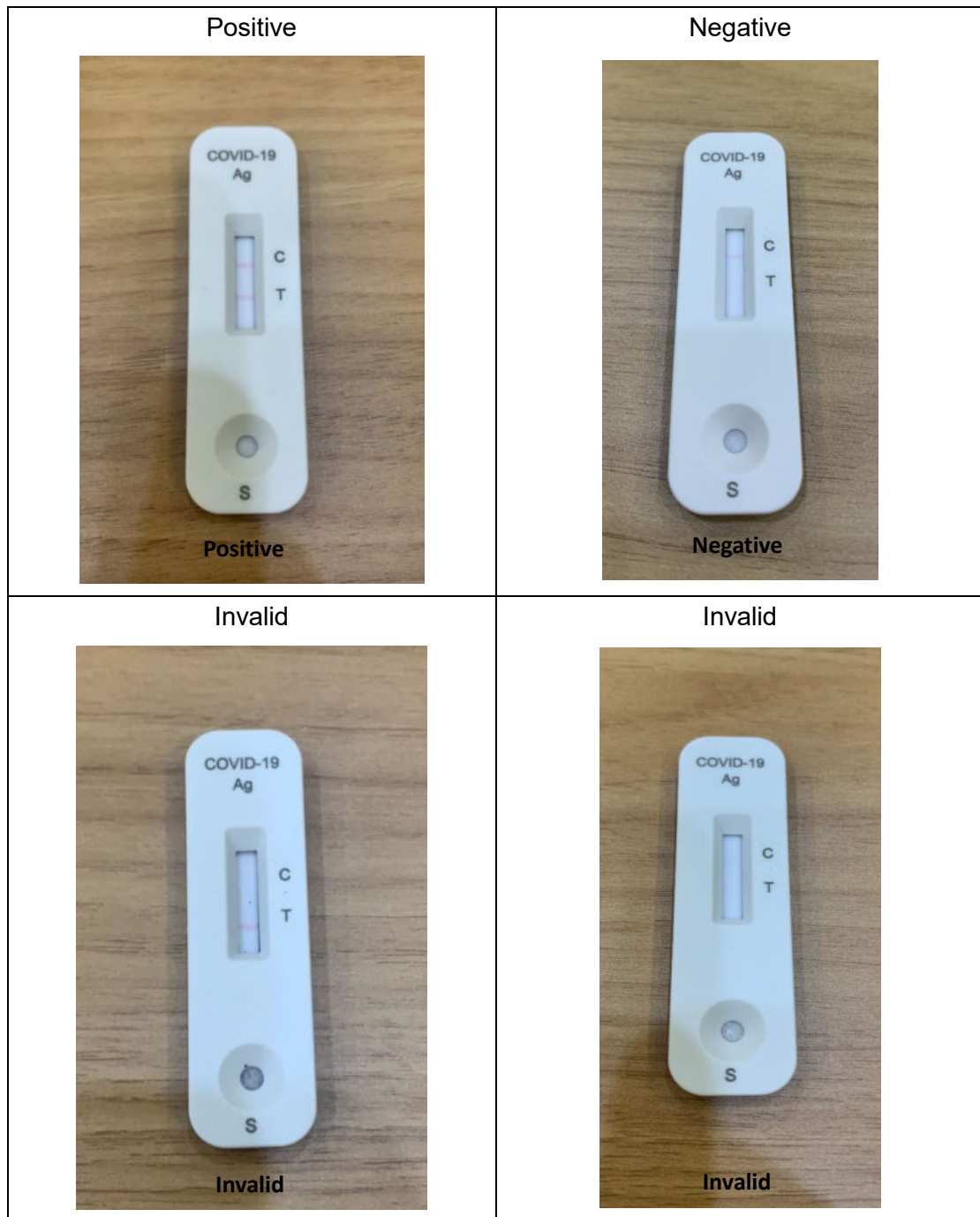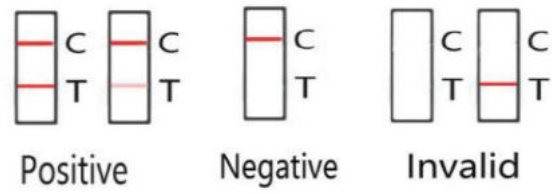

13. Do not read results beyond 20 minutes.

14. Change gloves before repeating the test.

Note: saliva/sputum sample should be collected after waking up and before drinking.